




**MIXPAC™**

# Intra Oral Tip

for 18 to 75 mL Systems

medmix offers a compatible range of accessories for static mixing tips. Snap-on Intra Oral Tips are used primarily for intraoral applications in confined areas to ensure easy, precise and safe dispensing of materials.

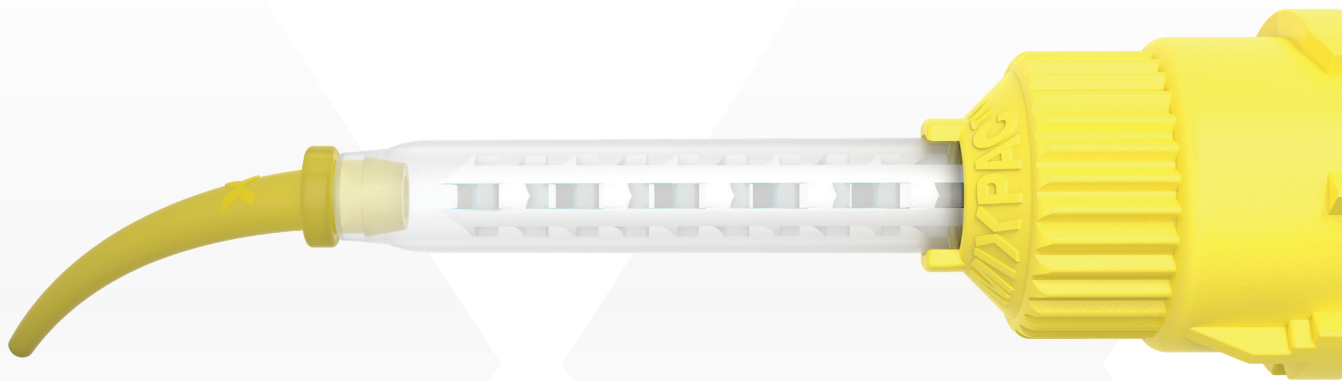

**Snap-on Intra Oral Tip facilitates easy, precise and safe dispensing of materials.**

Losing an Intra Oral Tip is one of the greatest risks during intraoral applications. Severe injury to the patient could occur if the application tip accidentally separates from the housing of the mixing tip and is swallowed by the patient resulting in serious legal consequences for the dentist and dental supplier.

MIXPAC's Intra Oral Tips are meticulously matched components, fully interchangeable and subjected to stringent quality-controlled testing to ensure safe and reliable compatibility between all components as well as an efficient time-saving clinical workflow.


- ✓ **MIXPAC's mixing tips are compatible** with MIXPAC's uniquely shaped Intra Oral Tips (IOT's) to fulfill the requirements for the MIXPAC seal of quality.
- ✓ **MIXPAC owns valuable trademark protection** for the shape, unique looking X marking and color of its products.
- ✓ All **MIXPAC Candy Colors™** are available upon request for Intra Oral Tips in any diameter, length and configuration for diverse material types and applications. The Candy Colors stand for our Swiss Quality.

# MIXAC™ Intra Oral Tip



## Low assembly force

The low force required to attach the Intra Oral Tip onto the housing of the mixing tip is minimized. The dimensions for individual components are inspected to guarantee a secure fit.

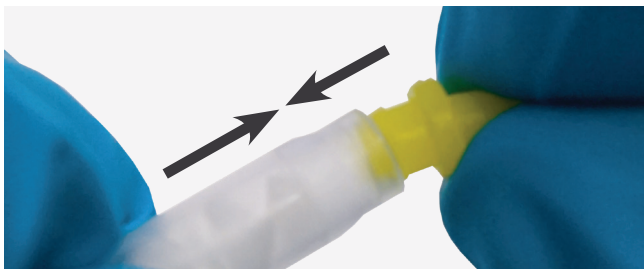

## Secure Position

The Intra Oral Tip remains securely positioned in place during dispensing of material through the mixing tip. It's optimal fit prevents the Intra Oral Tip from accidental rotation to avoid repeated applications; thus saving material and time.



Application of material around the preparation using the Intra Oral Tip.

## Secure Connection

The special design of the Intra Oral Tip ensures for a secure and tight connection between the housing of the mixing tip and Intra Oral Tip to prevent disengagement due to high extrusion forces.




Disengaged Intra Oral Tip during clinical application.

## Secure Flow

MIXPAC's Intra Oral Tips provide a consistent, constant flow of material during dispensing for precision application of material.

## Secure Sealing

MIXPAC's Intra Oral Tips are meticulously matched to the housing of the mixing tip to ensure for optimal tightness with leakage.

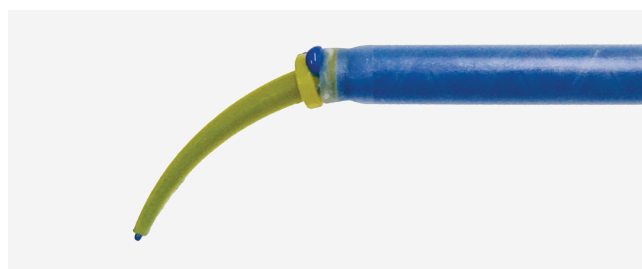

Leakage of material at the connection point between the Intra Oral Tip and the housing of the mixing tip

## Commitment to Quality, Regulatory & ESH

The Original Swiss Quality MIXPAC components are manufactured and certified according to ISO 9001 for industrial products and ISO 13485 for medical devices to ensure safe and reliable compatibility between all components as well as an efficient time-saving clinical workflow tightness with leakage.

- **ISO 13485** - Quality Management System
- **ISO 14971** - Product Risk Management
- **ISO 9** - Manufacturing environment
- **ISO 45001** - Occupational Health & Safety Management System
- **ISO 14001** - Environmental Management System
- **EU MEDICAL DEVICE REGULATIONS** - Technical Documentation



Inconsistent material flow (spiral jet) caused by a burr defect at the outlet of the Intra Oral Tip.

medmix Switzerland AG | Ruetistrasse 7 | 9469 Haag, Switzerland  
P +41 81 414 70 00 | [www.medmix.swiss](http://www.medmix.swiss)

Legal Notice: This brochure is a general presentation. The information contained in this publication is believed to be accurate and reliable but is not to be construed as implying any warranty or guarantee of performance. We waive any liability and indemnity for effects resulting from its application. Please, contact us for a description of the warranties and guarantees offered with our products. Directions for use and safety will be given separately. All information herein is subject to change without notice. MIXPAC, T-Mixer, Colibri and Candy Colors are trademarks of medmix Switzerland AG. medmix is a trademark of medmix AG. © medmix Switzerland AG 2023



 medmix

DENTAL